

## CPR

### Cardiopulmonary Resuscitation

1



Check for response

2



Call for help

3



Rate of 100 to 120 per min  
Depth 5~6cm

If needed begin CPR –  
give 30 chest compressions

4



Give two rescue breaths

5



Repeat CPR – continue to  
give 30 chest compressions  
and two rescue breaths

## AED

### Automated External Defibrillator

1




Open the AED cover  
This automatically turns the AED on

2



Remove the patient's clothes

3

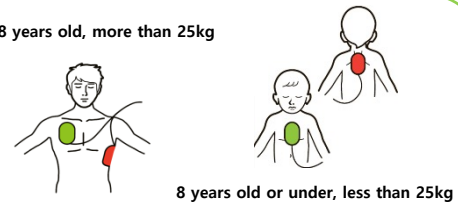


Open the pad pack, attach pads  
to the patient

4



When analysing heart rhythm,  
do not touch the patient

5

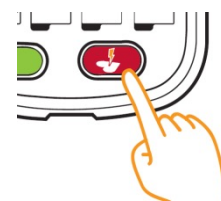

Press the flashing shock button  
when instructed